


Application and Fitting Guide

Heelift® AFO

- Choice of interior (smooth or convoluted)
- Made of latex-free polyurethane foam
- Single-patient use only.

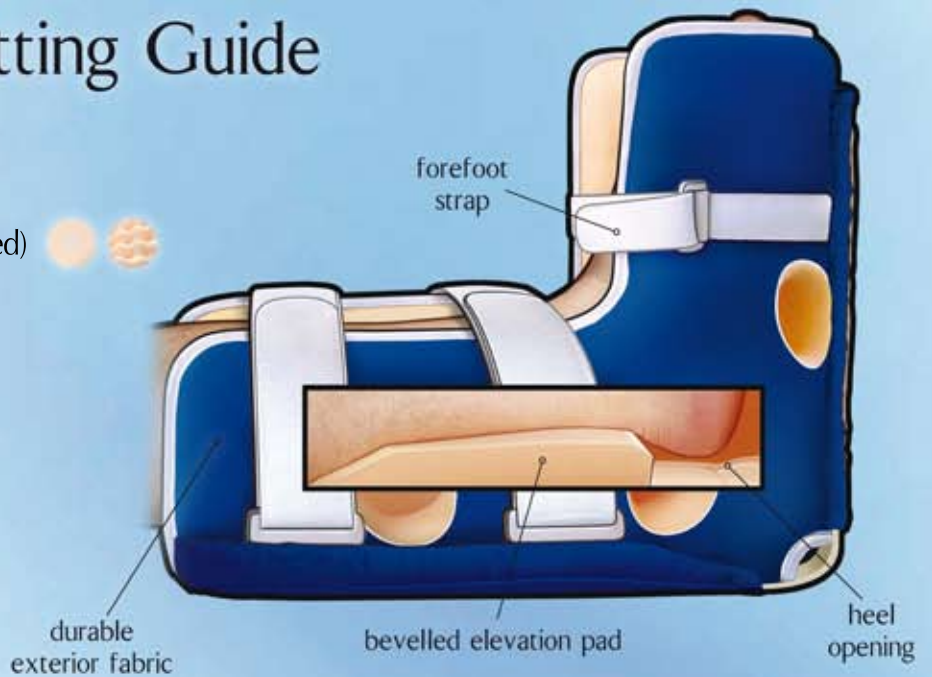

Easy Application and Fit

Available in multiple languages at:
<http://www.heelift.com/heelift-instructions.html>



1 Place foot inside boot with the heel above the opening.



2 Thread straps through D-rings. Secure the straps.



3 Test the fit under straps. **APPLY STRAPS LOOSELY.**



4 Test the fit under heel opening. Boot should not move on the leg.

Additional Features and Care



Semi-rigid brace keeps the ankle and foot in position.



Non-slip sole aids with assisted ambulation.

Heelift AFO is indicated for the following:

- Dorsiflexion ankle support for paralysed legs
- Comfortable night splint
- Treatment/Prevention of plantar flexion contracture
- Treatment of plantar fasciitis
- Treatment/Prevention of foot drop deformity
- Treatment of Achilles tendonitis

Patent Pending
 Heelift® AFO Authorized Code: L4396



Hand Wash



Machine Wash



Machine Dry



Autoclave



Sanitise

Optional Customisation

View customising details at:
<http://www.heelift.com/customize.html>

Foot Rotation



Hip Rotation



SCD with AFO



Achilles Tendon



Addl. Forefoot Protection



Malleolar Decubitus



INNOVATORS IN PRESSURE ULCER SOLUTIONS

Heelift® Suspension Boot
www.heelift.com



Heelift® Glide Boot
www.heeliftglide.com



Heelift® Traction Boot
www.tractionboot.com



Contact details:

Phone: 01986 798120 Fax: 01986 798040

Email: info@vmorthotics.co.uk Web: www.vmorthotics.co.uk